

EasySAFE[®] PLUS

SAFETY JAR

The new **EasySAFE[®] PLUS** safety jar is a medical device designed and manufactured for applications of “high flow and high vacuum” joint in use with the **EasyVAC[®] PLUS** vacuum regulators and with **EasyAIR[®]** “Venturi system” suction units.

This special safety jar is used to protect the equipment and the plant upstream, should the overflow valve in the main container not be working properly. The **EasySAFE[®] PLUS** safety jar is made of Polysulfone and fully autoclavable (134 °C - 18 min.), having an overflow floating valve and a housing designed to fit an antibacterial filter to guarantee the complete suction system protection against any contamination.

The collecting jar of this safety device has a deliberately reduced capacity (approx. 50 ml) so that even a small presence of fluid could immediately activate the floating valve to stop the suction.

The inlet connection with a quick release snap ring, the rolling 360° position of the patient outlet hose connector and the quick locking system jar-to-lid by 1/12 turn only make simple and easy the connection and removing of the **EasySAFE[®] PLUS** safety jar to the **EasyVAC[®] PLUS** vacuum regulators and **EasyAIR[®]** “Venturi system” suction units. Everything is made to make simple, fast, safe and easy the use by the hospital staff.



01



02




03

01 QUICK RELEASE SNAP RING FOR **EasyVAC[®] PLUS** AND **EasyAIR[®]** SUCTION DEVICES CONNECTION. PRESENCE OF SOFT GRIP INSERTS TO HELP THE HANDLING

02 360° ROLLING POSITIONING HOSE CONNECTOR $\varnothing = 8.0 \div 9.2$ MM

03 QUICK 1/12 TURN TO SPEED UP THE JAR CLEANING AND THE ANTIBACTERIAL FILTER REPLACEMENT (IF FILTER PRESENT)

04 HOUSING FOR PLACING AN ANTIBACTERIAL FILTER (OPTIONAL)

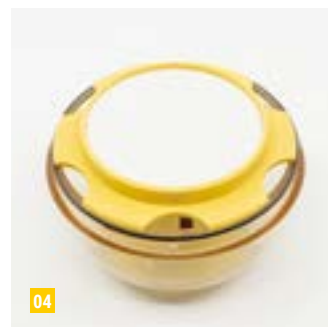

04

EXCELLENCE IN INNOVATION
BREAKTHROUGH TECHNOLOGY
INTELLIGENT DESIGN



flow-meterTM
ENGINEERING REVOLUTION IN MEDICAL TECHNOLOGY

REGISTERED OFFICE AND FACTORY ADDRESS: Flow Meter Spa - Via del Lino, 6 - 24040 Leivate - Bergamo - ITALY
Ph. +39 035 594047 | Fax +39 035 59482 | info@flowmeter.it | www.flowmeter.it

VAT. N. IT01737040160

EasyVAC® PLUS AND EasyAIR®
FITTED WITH EasySAFE® PLUS (OPTIONAL) ▶

- ▶ EasyVAC® PLUS VACUUM REGULATOR, PAGE 56
- ▶ EasyAIR® "VENTURI SYSTEM" SUCTION UNIT, PAGE 64



TECHNICAL SPECIFICATIONS | EasySAFE® PLUS

SIZES (LxWxH)	93x76x74 mm
WEIGHT	0.12 Kg
OVERFLOW SYSTEM	floating valve
PATIENT CONNECTION	hose connector Ø 8.0÷9.2 mm
MAXIMUM APPLICABLE VACUUM VALUE	-950 mbar / 5 min.